





Important! This quick reference guide does not replace the user manual which can be downloaded at www.humancaregroup.com. Read all instructions before use!

Components

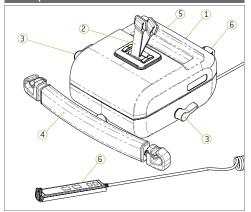

- 1 Secondaru Panel 2 Lift Belts
- 4 Telescoping Hanger Bar 5 Emergency Stop
- 3 Direction Levers 6 Hand Control

Secondary Panel Functions

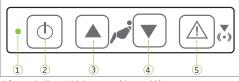

- 1 Status Indicator Light
- 2 Power On/Off
- 4 Lower Lift 5 Emergency Lowering
- 3 Raise Lift

Status Indicator Lights

Flashing Green On/Ready for Use Full Battery





Constant Green Lift in Use Lift Loaded




Constant Yellow Lift Charging Functions Blocked



Flashing Red/Yellow Service Passed Battery Low

Important! This overhead lift has been designed and tested for indoor use with one patient at a time. This product must not be immersed in water. Read all instructions for full product information!

Hand Control Functions

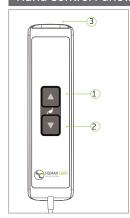

1 Up Button 2 Down Button 3 USB and Charge Port




Important!

The covers for the USB input and the charging input (3) must be closed for the hand control's water protection (IPX5) to be valid.

Charging the Lift

Flashing Red - Charge lift immediately!

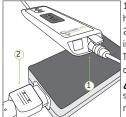

1 Connect charger plug to handset. 2 Connect plug to charger and

insert plug into wall to charge. The charger must only be connected to an earthed socket.

Note: If the emergency stop is activated, the lift will not charge!

Important! The lift can only be used with a Human Care charger and must be disconnected to use the lift.

Models: 55070 Human Care HC AB. All Rights Reserved.







Using the lift



Emergency Stop Activation

Press down on red button to activate

Emergency De-Activation Turn red button clockwise.


Important! The emergency lowering button must only be used in emergency situations.



Suspending the lift


Place the lift on a suitable surface and feed out belt by gently pulling out belt while at the same time pressing the "Down" button on either the hand control or secondary panel.

Hook lift belt onto distance strap so that it is held firmly in position (easiest to hook from above at an angle as shown). Ensure the lift belt is firmly in place before use.

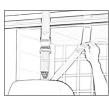

Room to room transfers

Activate desired belt by turning directional lever. If lever faces towards the leg hook, the belt nearest to the leg hook is active. If it faces towards the hanger bar, the belt closest to the bar is active.

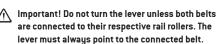



Slings

Attach head sling loops to carry bar first, then leg straps to leg hooks. Ensure the straps are secured beneath the safety pin.



Proper lifting techniques

Do not use the telescoping bar to move the patient and the lift. Do not hold onto the hooks while performing a lift. Once the patient is comfortably secured in sling, raise lift, hold sling and position/move patient.

General Care



Charge lift every night.



Wipe down with slightly dampened drying cloth.



Disinfect with DuPont™ Virkon®.



Never use strong detergents on lift.



Activate emergency stop when not in use.



Important! Do not use strap to transport!

Lift must always be stored or moved in horizontal position. Rest on cart to move.

Inspection

Annual Inspection - The lift must be inspected at least once per year by a gualified technician certified by Human Care. Monthly Inspection - Check lift belts for damage or wear. Feed entire length to ensure proper functioning. Daily Inspection - Always inspect the lift for visible signs of damage before use. Ensure the lift is correctly mounted on the distance strap or rail roller. Always check slings and belts

for signs of wear before use. Do not use if wear evident.

Troubleshooting




If the indicator light is not on, check the following:

- Mains switch is on
- Batteries do not require charging
- Emergency stop button is not pushed in (deactivate by turning clockwise)



If the indicator light is flashing, check the following:


- Check the hand control button functions up / down when depressed (replace if necessary)
- When pressing up, the lift is not already in its maximum upper position
- When pressing down, the lift is not already in its maximum lower position

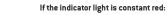

- The lift motor is overheated allow to cool
- The lift is overloaded lower using emergency lowering button





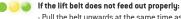

- Pull the belt upwards at the same time as you depress the Down button on the hand control / lift panel



If the indicator changes to flashing red:

- The batteries need charging, patient may be lifted a few more times

ROOMER S 220kg QUICK REFERENCE GUIDE